

FITTING*

1 Select cannula size

Select appropriate cannula size; A clear gap must be visible around each prong. The sizing chart on the following page should only be used as a guide.

2 Prepare skin

Prepare the patient's skin according to hospital protocol.

3 Connect to gas source

Connect the cannula to the gas source. TIP: Place hand close to nasal prongs to ensure that there is gas flow exiting the prongs.

4 Remove the first F&P Wigglepad™ 2 tabs

Remove the first backing tabs from the F&P Wigglepads™ 2 and avoid touching the adhesive.

5 Insert cannula


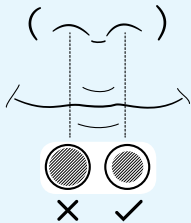




Insert the cannula into the nares. Ensure the cannula bridge rests close to the nose without touching the septum. DO NOT stretch the cannula during application. Stick the F&P Wigglepads™ 2 to the patient's cheeks.

6 Secure cannula

Remove the second backing tabs and stick the F&P Wigglepads™ 2 onto the cheeks.

* IMPORTANT: Always refer to the user instructions supplied with the product for full set up instructions, warnings, cautions and contraindications.

1 SELECT CANNULA SIZE

SIZE RANGE		ENSURE GAP AROUND PRONGS*
XS	 OJR410	
S	 OJR412	
M	 OJR414	
L	 OJR416	
XL	 OJR418	



REMOVE CANNULA

1 Place fingertip

Place fingertip on the outside edge of the F&P Wigglepads™ 2 and gently peel the cannula away from the Wigglepads 2.



F&P WIGGLEPADS™ 2 REMOVAL AND REPLACEMENT

1 Lift the edge

Lift the edge of the F&P Wigglepads™ 2. Use a damp cloth to wipe the patient's skin and the underside of the Wigglepads 2 while gently peeling away from the patient's face.



2 Adhere






Adhere the replacement F&P Wigglepads™ 2 to the cannula, remove the first backing tabs and stick the Wigglepads 2 onto the patient's cheeks.



3 Remove

Remove the second backing tabs and stick onto the patient's cheeks.

SIZING INFORMATION

		WEIGHT GUIDE (KG)								SPARES							
		500g	1	1.5	2	2.5	3	3.5	4								
	OJR410 XS	[Fit chart for OJR410 XS]								F&P Wigglepads™ 2 WJR110							
	OJR412 S																
		WEIGHT GUIDE (KG)												SPARES			
		1	2	3	4	5	6	7	8	9	10	15	20	25	30		
	OJR414 M	[Fit chart for OJR414 M, OJR416 L, OJR418 XL]												F&P Wigglepads™ 2 WJR112			
	OJR416 L																
	OJR418 XL																
		 Expected to fit patient May fit patient															

NOTE:

- Prongs must not create a seal in the nares. A clear gap must be visible around each prong.
- Patient weight should only be used as a guide.



To view the Optiflow™ Junior 2 inservice video scan this QR code

* IMPORTANT: Always refer to the user instructions supplied with the product for full set up instructions, warnings, cautions and contraindications.